

Absolute Athletics



Heel Support Pad

5 in 1 Foot & Ankle Orthosis

Designed to help relieve posterior heel pain, the Orthotic device applies counterforce pressure on the Achilles Tendon.

Indications for use:

1. Achilles' Tendonitis - Place pad in superior position.
2. Haglund's Disease - Pump bump deformity, place pad in inferior position.
3. Sever's Disease - Traction epiphysitis of the calcaneus, place pad in inferior position.
4. Heel Spur - Invert neoprene product and place pad beneath heel, with horseshoe open towards toes.
5. Plantar Fasciitis - Invert neoprene product and place pad appropriately between arch and heel.

1



2 & 3



4



5



Bunga Pads™